# User Safety Manual

## Instructions for Use

THIS USER MANUAL MUST BE READ BEFORE USE

### ⚠ Contraindications

- Do not use if you are or may be pregnant. It is not known whether TENS may affect foetal development.
- Do not use if you have a pacemaker (or if you have a heart rhythm problem) or with any electronic medical devices. Using this unit with electronic medical devices may cause erroneous operation of the device. Stimulation in the direct vicinity of an implanted device may affect some models.
- Do not use following a pelvic surgery or vaginal childbirth until you are fully healed. Stimulation may disrupt the healing process.
- Do not use if you have symptoms of active urinary tract infection, vaginal infections, or localised lesions. Introducing the probe may irritate sensitive tissue.
- Do not use if you have poor sensation in the pelvic region. You may not be able to control the intensity of stimulation safely.

### ⚠ Warnings

- Do not use if you are unable to properly insert the probe. If you have a severe prolapse, or if any discomfort occurs when inserting the probe, consult your healthcare professional before use.
- Do not use when driving, operating machinery, or similar actions needing muscular control. Loose electrodes, damaged leads, or sudden changes in contact may cause brief involuntary muscle movements.
- Do not use to mask or relieve undiagnosed pain. This may delay diagnosis of a progressive condition.
- Do not use if you have, in the area being treated: active or suspected cancer or undiagnosed pain with a history of cancer. Stimulation directly through a confirmed or suspected malignancy should be avoided as it may stimulate growth and promote spread of cancer cells.
- Do not use electrodes on the front of the neck. Stimulation on the front of the neck can affect your heart rate or cause contraction of the throat.
- Do not use electrodes across the chest as this may increase the risk of cardiac fibrillation. Very strong stimulation across the chest may cause an extra heartbeat and/or rhythm disturbances to your heart, which could be lethal.
- Do not use simultaneously with high frequency hospital diagnostic/therapeutic equipment. Doing so may result in burns at the site of the electrodes and possible damage to device.
- Do not use the device in close proximity (e.g. 1M) to a shortwave or microwave as this may produce instability in the stimulator output.

- Stimulation should not be applied across or through the head, directly on the eyes, covering the mouth, on the front of the neck, or from electrodes placed on the upper back or crossing over the heart.
- Stop using the device and consult a health care professional if you experience any adverse reactions from the device.

### ⚠ Cautions

- Insertion of the probe makes it unsuitable for use in children without clinical supervision.
- Caution should be used if you have a bleeding disorder as stimulation may increase blood flow to the stimulated region.
- Caution should be used if you have suspected or diagnosed epilepsy as electrical stimulation may affect seizure threshold.
- Caution should be observed when using the device when connected to monitoring equipment with body worn electrodes. It may interfere with the signals being monitored.
- Do not permit use by children unable to understand the instructions or persons with cognitive disabilities, i.e.; Alzheimer's disease or dementia.
- Keep away from children under 5 years of age, pets and pests. Long lead wires and charging cables - risk of strangulation in infants.
- Do not fall asleep whilst using the device.
- Caution should be observed when using the device at high strength settings. The device has Yellow LED lights on output sockets which means the output will exceed 10 mA (R.M.S) or 10 V (R.M.S) averaged over any period of 1 sec. Prolonged use at high settings may cause muscle injury or tissue inflammation.
- Not intended for use in an oxygen rich environment.
- Not intended for use in conjunction with flammable anaesthetics or flammable agents.
- The patient is an intended operator.
- Do not service and maintain the device while in use.
- Maintenance and all repairs should only be carried out by an authorized agency. The manufacturer will not be held responsible for the results of maintenance or repairs by unauthorized persons. If necessary, we will provide circuit diagrams, component part lists or other information that will assist authorized service personnel to repair the device.
- The operator should not touch the patient at the same time when touching the battery output.
- Do not immerse your device in water or place it close to excessive heat such as a fireplace or radiant heater or sources of high humidity such as a nebulizer or kettle as this may cause it to cease to operate correctly.
- Keep the device away from sunlight, as long-term exposure to sunlight may affect the rubber causing it to become less elastic and crack.
- Keep the device away from lint and dust, as long-term exposure to lint or dust may affect the sockets or cause the battery connector to develop a bad contact.
- There are no serviceable user parts. Do not attempt to open or modify the device. This may affect the safe operation of the unit and will invalidate the warranty.
- Only use the device with the accessories provided with this kit. Using other accessories may expose user to risk or reduce treatment effectiveness.
- Allow at least 30 minutes for the device to warm up or cool down from its minimum or maximum storage temperature before it is ready for use.
- Note: No serious or long-term adverse effects have been reported. Mild adverse reactions are very rarely reported, but these have included muscular pain and cramps, vaginal tenderness, irritation and bleeding, mild or short-term urge or faecal incontinence, and tingling sensation in legs. If you experience any of these, stop use immediately. When symptoms have gone, try resuming at a lower intensity setting.
- Note: The electrical specifications are nominal and subject to variation from the listed values due to normal production tolerances of at least 5%.

### ⚠ Probe Cautions

- The probe is intended for single patient use only. Do not share the probe with anyone else. Improper treatment or

### 🇬🇧 Cross-infection may occur.

- It is important the oboe is cleaned after each use. Ineffective cleaning may lead to irritation or infection.
- Never insert or remove the probe unless the control unit is off as insertion or removal when stimulation is active may cause discomfort or tissue irritation.
- If tissue irritation occurs, discontinue treatment immediately. Ask your healthcare professional for advice before continuing further treatment to prevent injury.
- Do not use a silicone based lubricant on the metal plates of the probe as it may decrease the effectiveness of the device's muscle stimulation.
- Do not use this device with any probe other than those recommended by the manufacturer. Electrodes with smaller surface area may cause tissue irritation.
- The probe should be replaced every 6 months to ensure hygiene. Do not use if broken, damaged or showing signs of degradation.

### ⚠ Electrode Cautions

- Do not ignore any allergic reaction to the electrode: If a skin irritation develops, stop using the device, as this type of electrodes may not be suitable for you. Alternative electrodes specially made for sensitive skin are available.
- Do not use this device with leads or electrodes other than those recommended by the manufacturer. Performance may vary from specification. Electrodes with smaller surface area may cause tissue irritation.
- Do not use high intensity settings if electrodes are smaller than 50x50mm.
- Do not use electrodes if they have lost their adhesion. Treatment effectiveness may be reduced.
- Electrodes supplied have no measurable latex content.

#### Do not place electrodes:

- On skin, which does not have normal sensation. If the skin is numb too great a strength may be used, which could result in skin inflammation.
- On broken skin, electrodes could encourage infection.
- On the front of the neck. This could cause the airway to close, giving breathing problems. May cause sudden drop in blood pressure (vasovagal response).
- Over the eyes. May affect eyesight or cause headaches.
- Across the front of the head. Effect on patients who have had strokes or seizures is not known. May affect your sense of balance. The effects of stimulation on the brain are unknown.

### EMC

Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies can affect this equipment and should be used no closer than 30 cm (12 inches) to any part of the device.  
Note: As indicated in 5.2.1.1(f) of IEC 60601-1-2 for ME EQUIPMENT. Note: For hospital use, full EMC advice tables are available on request.

### Disposal

One of the provisions of the European Directive 2002/96/CE is that anything electrical or electronic should not be treated as domestic waste and simply thrown away. To remind you of this Directive all affected products are now being marked with a crossed-out wheellie bin symbol.  
To comply with the Directive, you can return your old electro-therapy unit to us for disposal. Simply print a postage-paid PACKETPOST RETURNS label from our website [www.tenscare.co.uk](http://www.tenscare.co.uk), attach this to an envelope or padded bag with the unit enclosed, and post it back to us. Upon receipt, we will process your old device for components recovery and recycling to help conserve the world's resources and minimise effects on the environment.

### Warranty

This warranty refers to the unit only. It does not cover the probe, electrodes, leads or any other accessory. This product is warranted to be free from manufacturing defects for 2 years from date of purchase. This warranty is void if the product is modified or altered, is subject to misuse or abuse; damaged in transit; lack of responsible care; is dropped; if incorrect battery has been fitted; if the unit has been immersed in water; if damage occurs by reason of failure to follow the written instructions for use booklet enclosed; or if product repairs are carried out without authority from TensCare. We will repair, or at our option replace free of charge, any parts necessary to correct material or workmanship, or replace the entire unit and return to you during the period of the warranty. Otherwise, we will quote for any repair which will be carried out on acceptance of our quotation. The benefits conferred by this warranty are in addition to all other rights and remedies in respect of the product, which the consumer has under the Consumer Protection Act 1987. Our goods come with warranties that cannot be excluded under the UK consumer Law. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality.

#### Before you send your unit for service

Read your manual and make sure you follow all the instructions for use.

#### Returning your unit for service

Should repair be needed within the warranty period, enclose your proof of purchase receipt. Please ensure all relevant details are completed before sending your unit in for service. Please ensure your contact details are still current and include a brief description of the problem you are experiencing together with your purchase receipt. For hygiene reasons, please do not include used electrodes. Send only the device.

Please return the unit at your cost to: TensCare Ltd, 9 Blenheim Road, Epsom, Surrey, KT19 9BE, UK If you are experiencing issues with the device and are not able to resolve the issue using the troubleshooting section of the device manual, or to report unexpected operation or events, or to provide feedback call TensCare or your local supplier or distributor.  
Contact TensCare customer service on +44 (0) 1372 723 434. Our staff are trained to assist you with most issues you may have experienced, without the need to send your product in for service.

European Medical Device Regulation requires that any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority in your country. This can be found at: <https://ec.europa.eu/docsroom/documents/36683/attachments/1/translations/en/renditions/pdf>

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### Entsorgung

One of the provisions of the European Directive 2002/96/CE is that anything electrical or electronic should not be treated as domestic waste and simply thrown away. To remind you of this Directive all affected products are now being marked with a crossed-out wheelee bin symbol. To comply with the Directive, you can return your old electro-therapy unit to us for disposal. Simply print a postage-paid PACKETPOST RETURNS label from our website www.tenscare.co.uk, attach this to an envelope or padded bag with the unit enclosed, and post it back to us. Upon receipt, we will process your old device for components recovery and recycling to help conserve the world's resources and minimise adverse effects on the environment.

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