Electrodes

Face 30x50mm

4 Reusable Face Electrode Pads

REF E-CM3050

Instructions For Use

Please read carefully before use

TensCare™

Instructions for Use

Use with an EMS unit conforming to EN60601-2-10 such as **perfect BEAUTY** or **Sports TENS**.

- 1) Before use make sure your skin is clean and dry.

Do not place electrode pads on skin, which does not have normal sensation or on broken skin.

- 2) Push the pin ends of the lead wires firmly into the pigtail ends of the electrode pads.

Note: A single face electrode pad acts as a pair of TENS electrodes. Connect both pigtail leads to the same TENS lead.

- 3) Peel the electrode pads from their protective plastic shield by holding and lifting one corner of the pad and pulling.


Do not pull on the pigtail wire of the pad.
Always check that the unit is **OFF** before applying or removing electrode pads

- 4) Apply the adhesive side of the electrode pads as shown in the picture.
Attach with the lead with red connector closest to the ear.
Choose a muscle toning programme using between 40 and 60 Hz, and between 250 and 350 μ s.

WARNING: Pads are one fifth of the size of standard TENS pads. Use a much lower intensity than you would with standard pads, and adjust carefully. Excessive current density can cause skin damage. We recommend that only stimulators with 1mA adjustment steps be used.

- 5) Remove the electrode pads from your skin by holding the pad itself and gently pulling.
- 6) After use always replace the electrode pads on the plastic liner and replace them in the re-sealable plastic bag.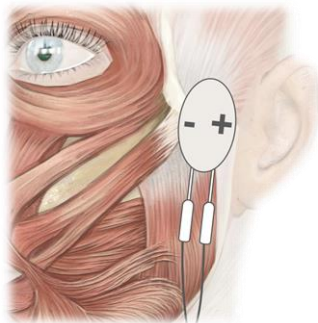

Important: Observe all the Contraindications, Cautions and Warnings included with the device that you are using with these electrodes.

Note: The + electrode pad needs to be plugged in to the red lead wire connector, the – refers to the one for the black lead wire connector.

Advice

- The electrode pads supplied are reusable but for single patient use.
- The electrode pads provided are latex-free.
- To obtain the best conductivity through the electrode pads always ensure that they are in good condition and tacky.
- Replace the electrode pads when they lose their stickiness or dry out.
- In very hot weather the gel on the electrode pads may become soft.

Storage life

Electrode pads should last 12 to 20 applications, depending on skin condition and humidity. Storage life of an unopened pack of electrode pads is 3 years. This may be affected by very high temperatures or very low humidity.

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TensCare Ltd.
Advena Ltd.